

# ASM

## 200

User Manual



# Contents

<b>Components</b>	3
-------------------	---

---

## **H/W Parts**

Device Functions	4
Power On/Off	5
Battery / Status LED Indication	6

---

## **System Settings**

Date & Time	7
Wi-Fi	9

---









## **Application**

Application Settings	10
Customer	11
Switching lens Hair Analysis (ASM-202 Lens)	12
Hair Loss Status	13
Scalp Status	14
Hair Density	15
Keratin of Scalp	16
Scalp Sensitivity	17
Switching lens Hair Analysis (ASM-203 Lens)	18
Hair Thickness	19
Hair Pore Status	20
Switching lens Hair Analysis (ASM-204 Lens)	21
Cuticle Status	22
Summary	23
Send Result -Email	24
Mirroring	25
View Mode	27


---

<b>Warranty</b>	28
-----------------	----

# Components

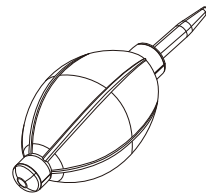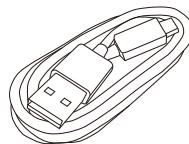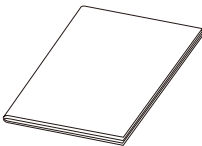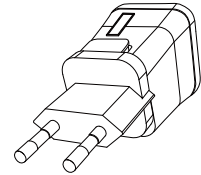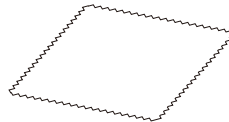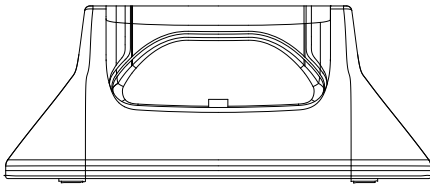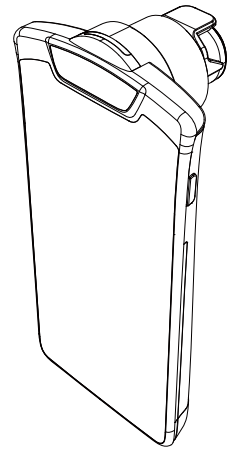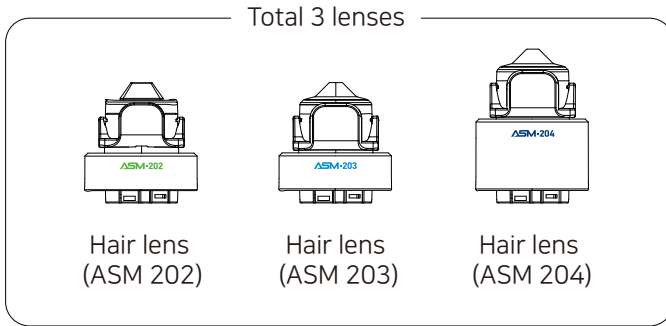


# Device Functions

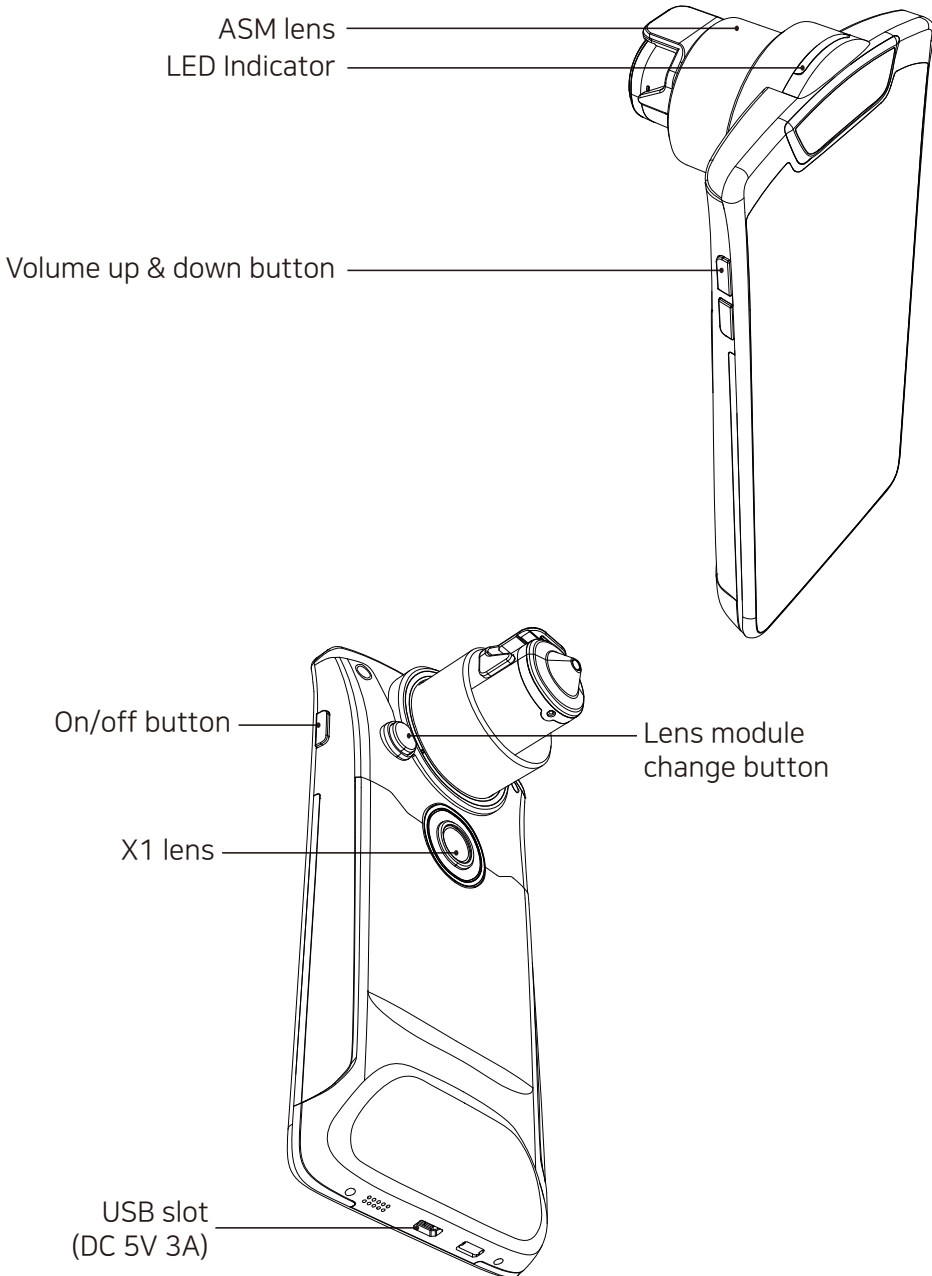

## Power On/Off


To turn on & off the power

- ① Press the power button and hold.
- ② Then "LED Indicator" will be twinkle and turn to white.

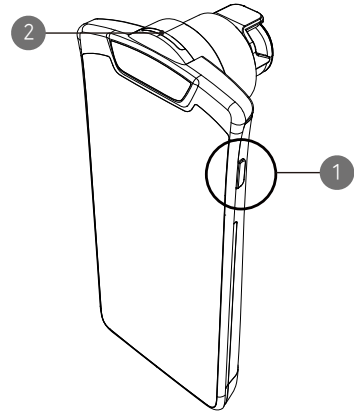


Charger can be used for American type & European type. The default is European type.


- ① Push the plug up.

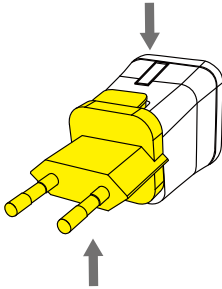


- ② Then European type is disassembled and you can see the American type inside of the charger.

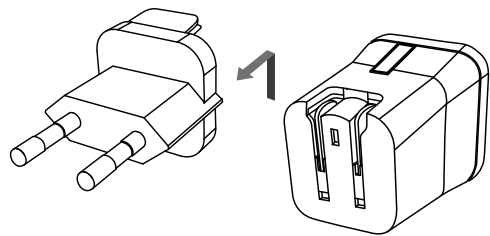

- ③ Choose the one you want to use and connect USB cable in adaptor.

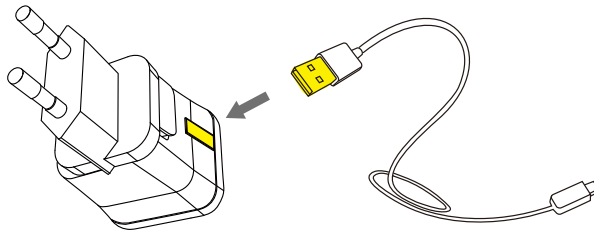


# Battery / Status LED Indication

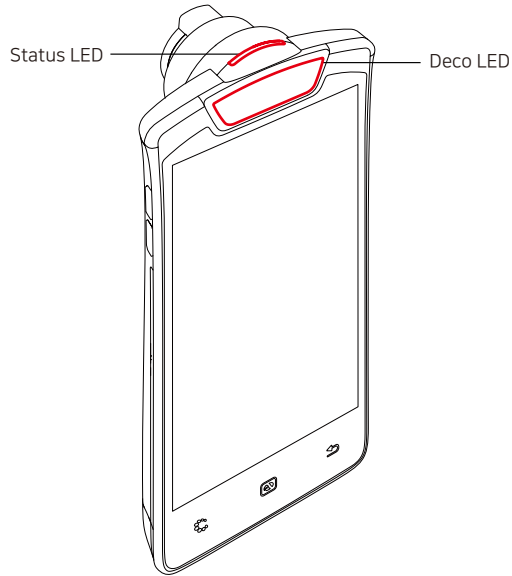

CAMERA POSITION	STATUS	DECO LED	STATUS LED	
In use	While turning it on	Light on	Blue light	●
In use	When running the app	Light on	Off	●
In use	Low battery	Displayed only on the indicator at the top of the LCD screen		
Plugged to the base	Charging	Off	Red light (When the LCD screen is off)	●
Plugged to the base	Fully charged (180 minutes)	Off	Blue light (When the LCD screen is off)	●


# Date & Time




1 Press  button to return to homepage.




2 Touch  button.

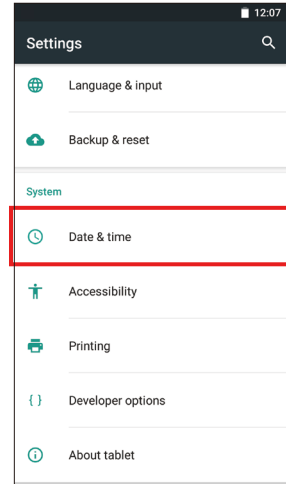


3 Go to "Date & time"

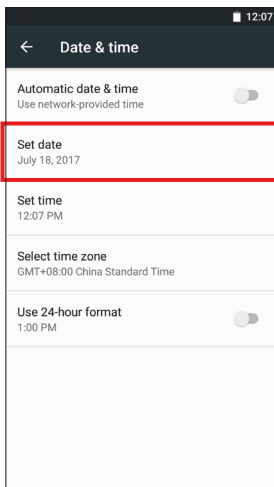

4 Go to "Set date".

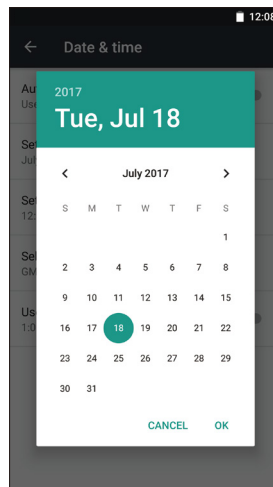


5 Select the date and touch "OK" button.

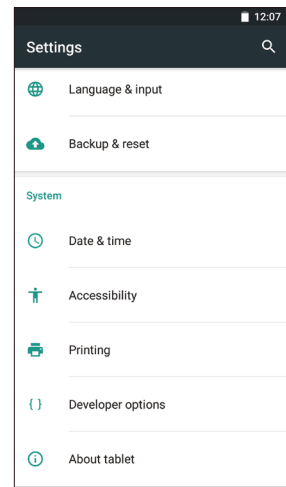


6 Check if the setting is correctly applied.


# Date & Time

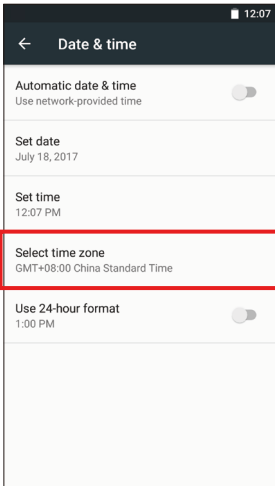


7 Touch "Select time zone"

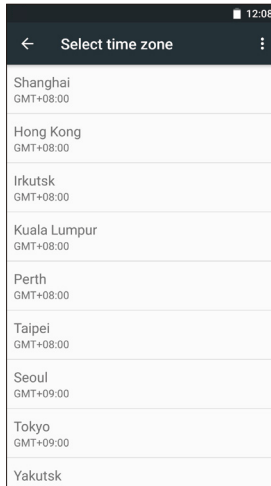


8 Select proper time zone.

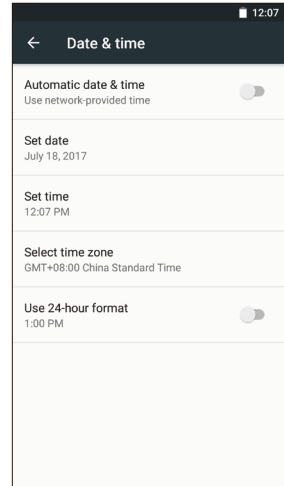


9 Check if the setting is correctly applied.

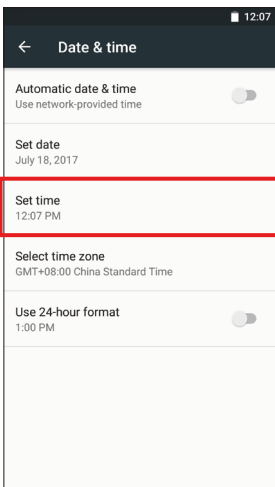

10 Touch "Set time".

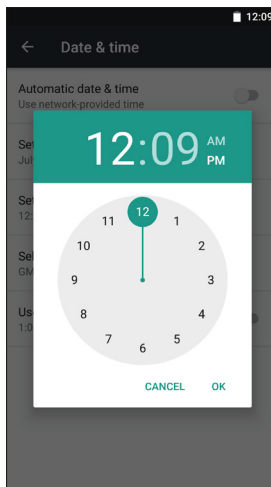

11 Change into the current time and touch "OK" button.

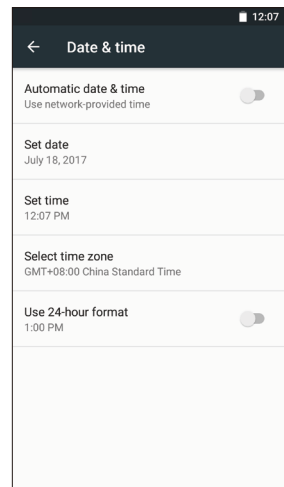


12 Check if the setting is correctly applied.




# Wi-Fi




1 Touch  button.

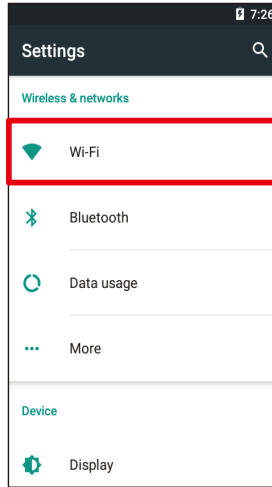


2 Touch "Wi-Fi".

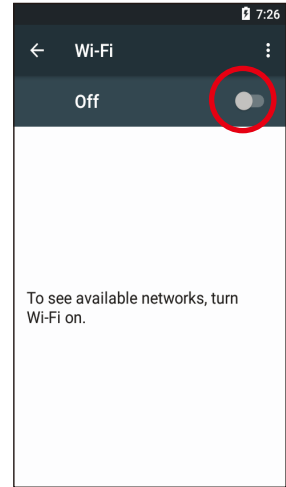


3 Turn on toggle switch.

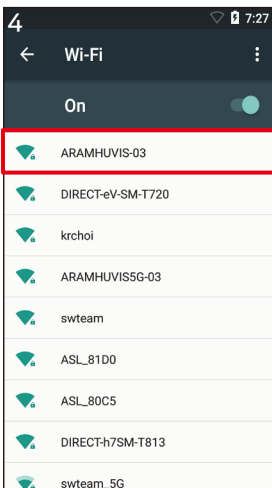

4 Select Wi-Fi

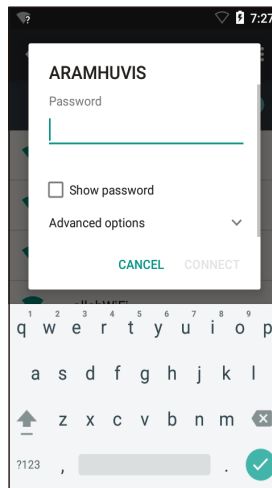

5 Among the lists, select the Wi-Fi connection you could use.

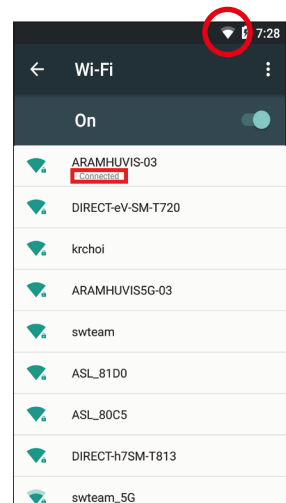


6 Wait until the device is connected to Wi-Fi.


# Application Settings




① Touch  button.

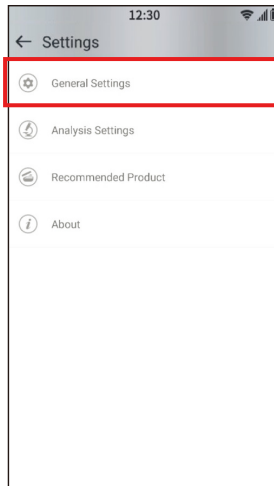


② Go to "General Settings".

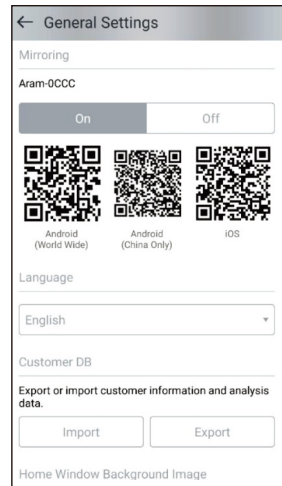


③ Available to change Language, Version and Email setting.

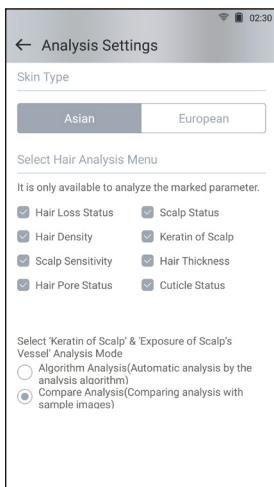

④ Select skin type and analysis criteria at skin analysis setting.

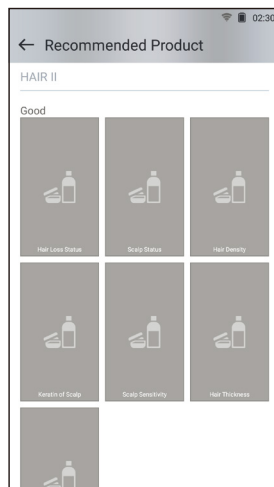

⑤ Register corresponding products at recommended product setting.

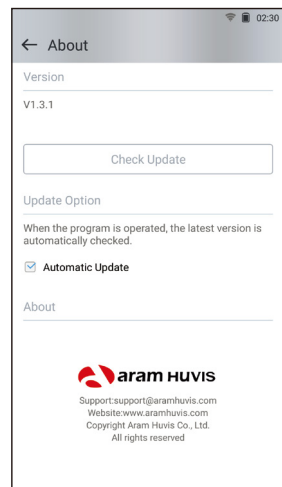


⑥ Proceed update at information.


# Customer




① Touch "Customer".

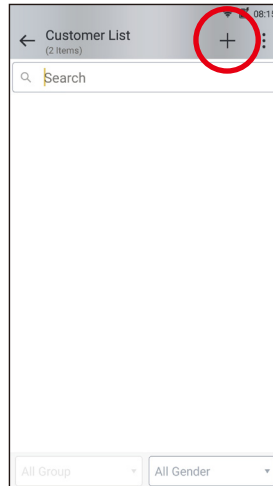


② Touch + to register new customer.

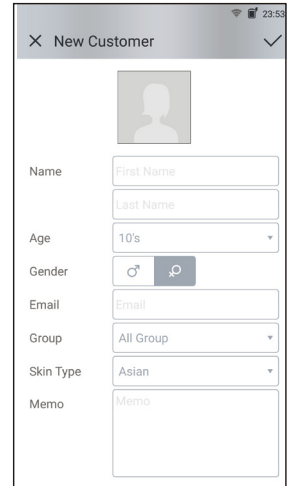


③ Input the information.

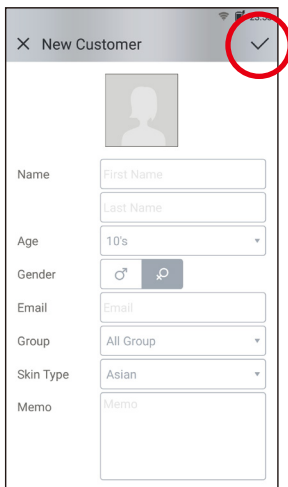

④ After putting all information, touch ✓.

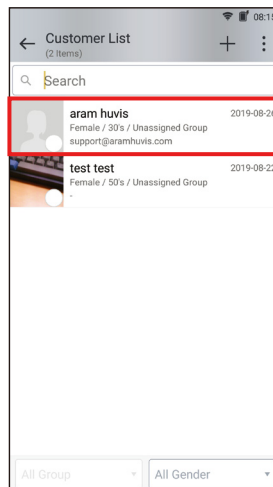


⑤ Select created user profile.

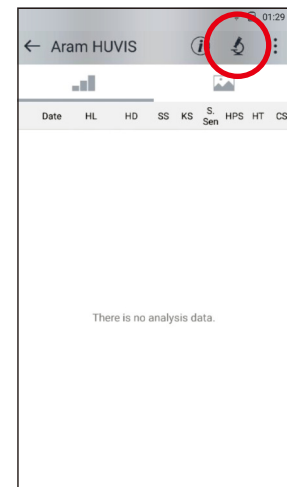


⑥ Touch  button to start analyzing.

# Taking off the lens ASM-202

- 1 Press switching button located below the lens for switching the lens.

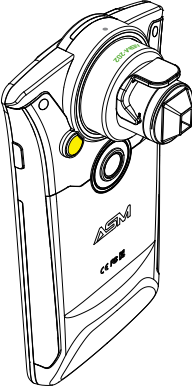


- 2 Turn the lens about 45 degrees counter-clockwise while pressing the switching button to take off the lens.

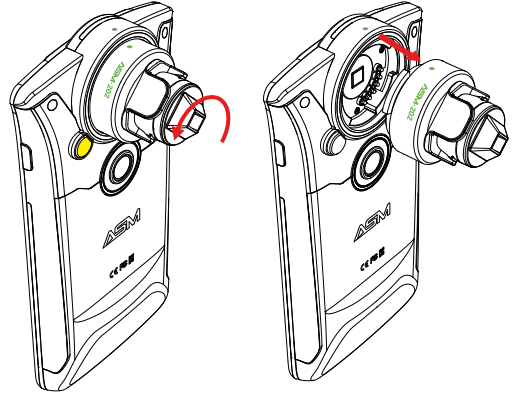


# Inserting lens

- 1 Until the correct lens is inserted, lens indicating label will blink.

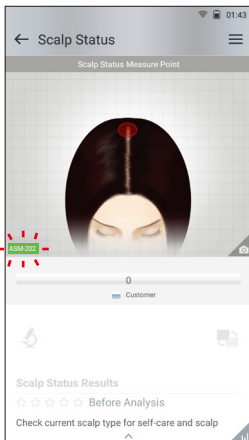


- 2 To insert the lens, match the red dot on lens and gray dot on device, turn it clockwise until clicks.

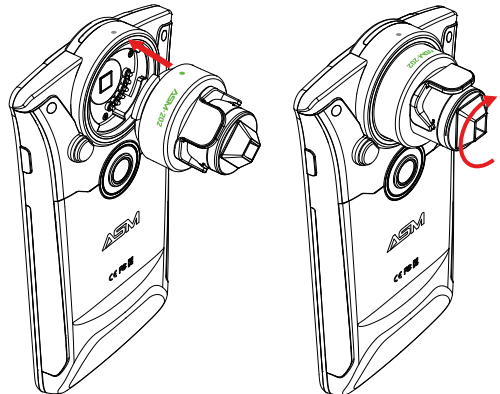


# Hair Loss Status X1 Lens




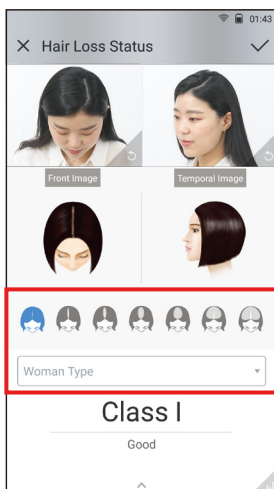
① Touch  button.

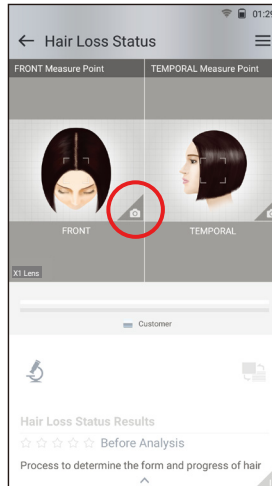

② Check capturing part and press  button to get into capture mode.

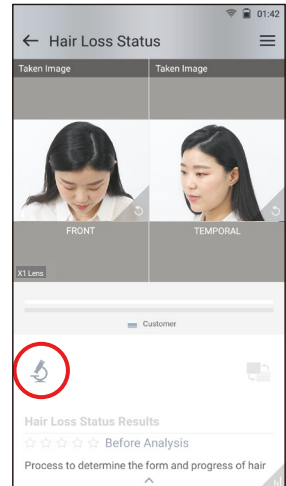



③ Capture front & side images and click  for getting into analysis mode




④ Select hair loss type and progress similar to captured images.

\*Only "Woman Type" is selectable for woman

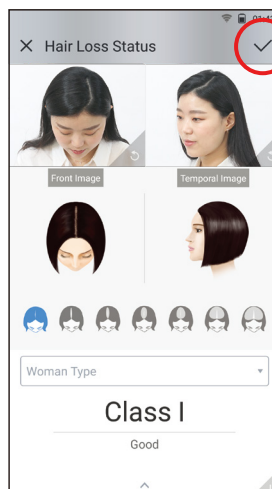


⑤ Click  to finish analysis and save.

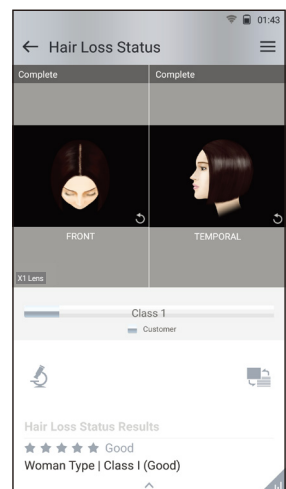

⑥ Result will appear. Press button to compare against sample.


# Scalp Status ASM-202

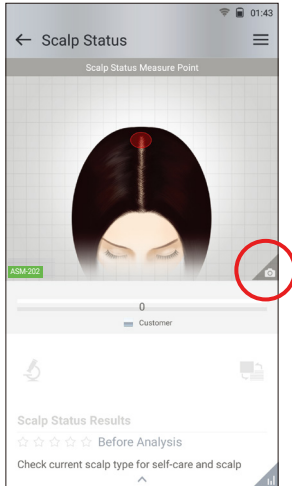


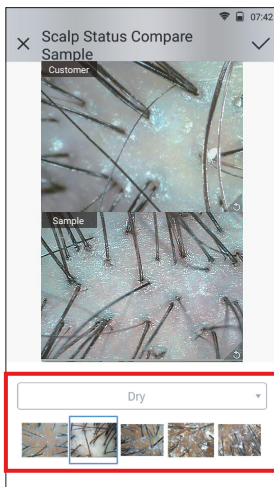
1 Check capturing part and press  button to get into capture mode.




2 Focus on image which you wish to capture.

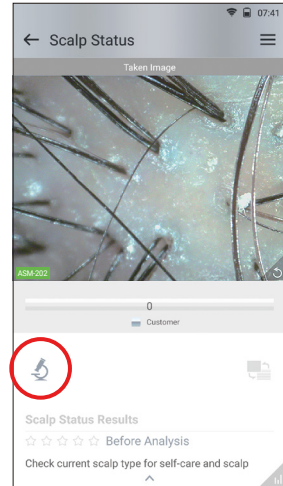



3 Press  button to analyze.



4 Select type and sample image similar to captured image.

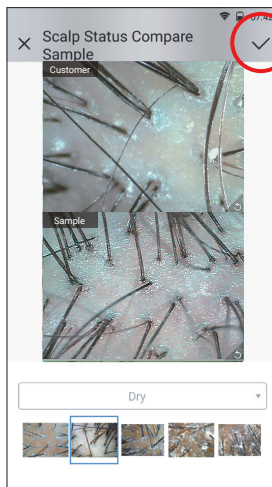


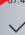5 Click  to finish analysis and save.

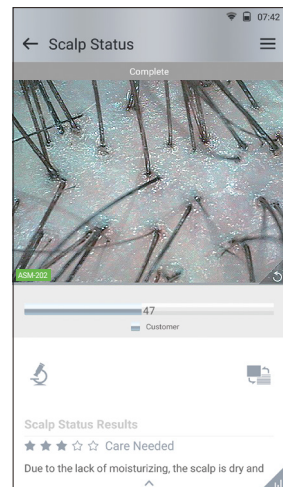


6 Check the result. To move on next page, slide the screen from right to left.


# Hair Density ASM-202

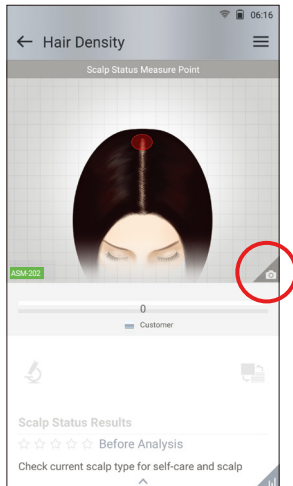


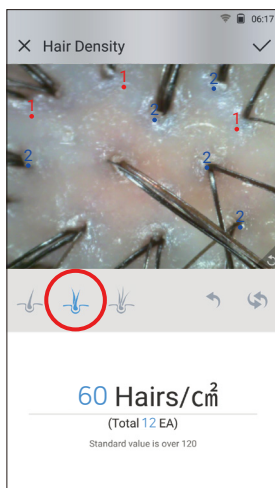
1 Check capturing part and press  button to get into capture mode.

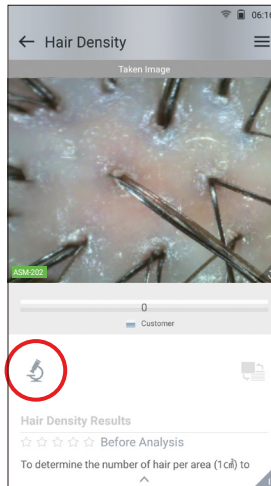


2 Focus on image which you wish to capture.

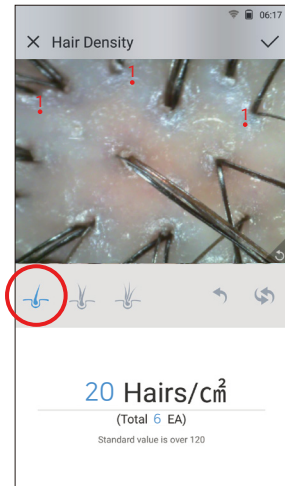



3 Select  and touch each and every pore that has a single hair from the captured image.



4 Similarly, do the same for pores with two hairs and three hairs.

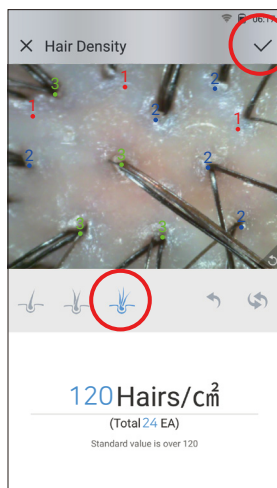


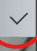5 Click  to finish analysis and save.

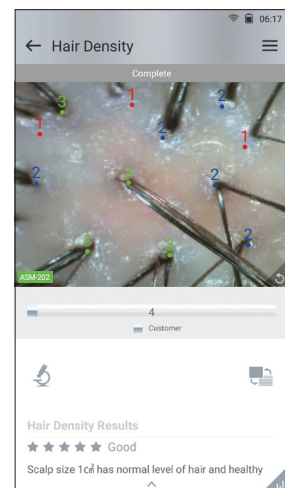

6 Check the result. To move on next page, slide the screen from right to left.


# Keratin of Scalp ASM-202

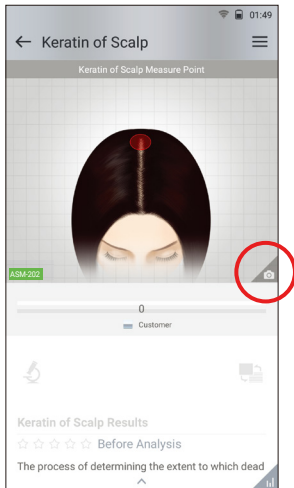


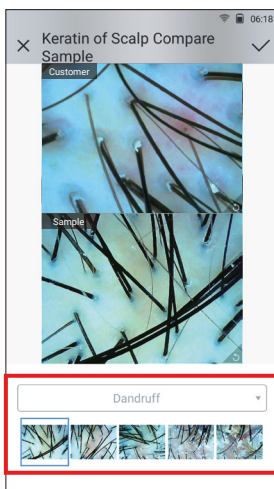
1 Check capturing part and press  button to get into capture mode.




2 Focus on image which you wish to capture.

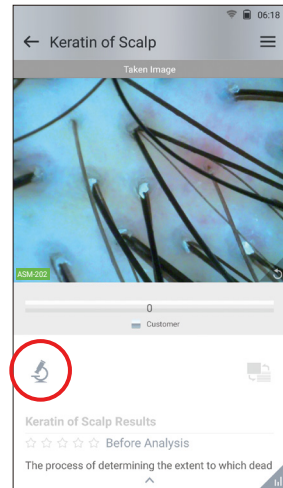



3 Press  button to analyze.



4 Select type and sample image similar to captured image.

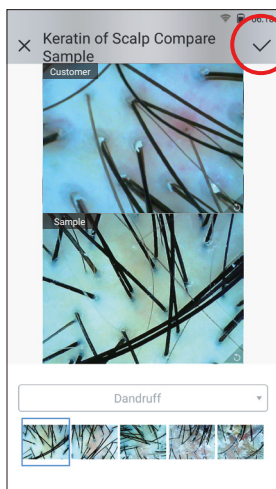

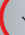5 Click  to finish analysis and save.

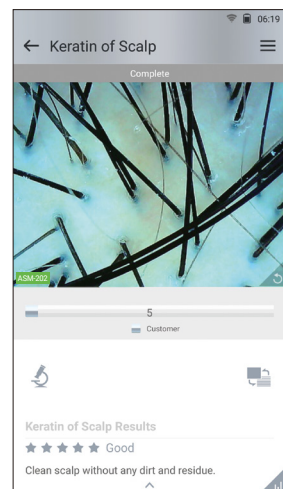


6 Check the result. To move on next page, slide the screen from right to left.




# Scalp Sensitivity


ASM-202

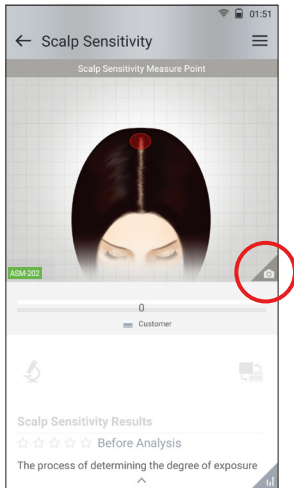


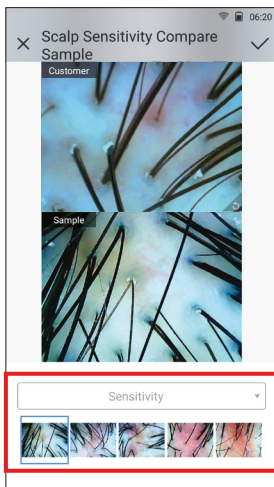
1 Check capturing part and press  button to get into capture mode.

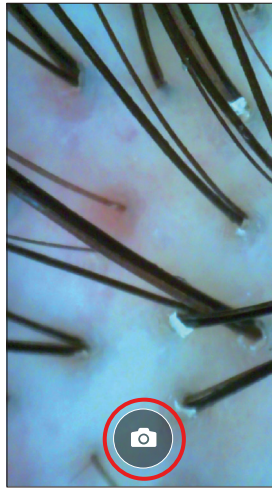


2 Focus on image which you wish to capture.

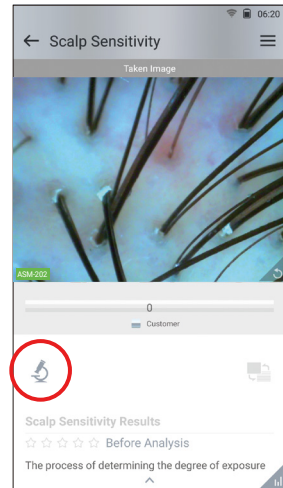



3 Press  button to analyze.



4 Select type and sample image similar to captured image.

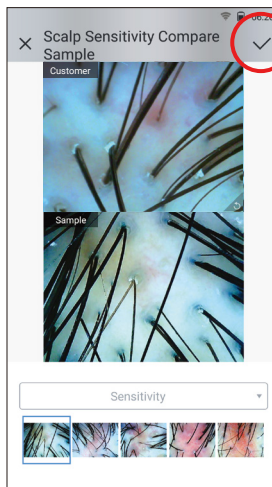

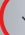
5 Click  to finish analysis and save.

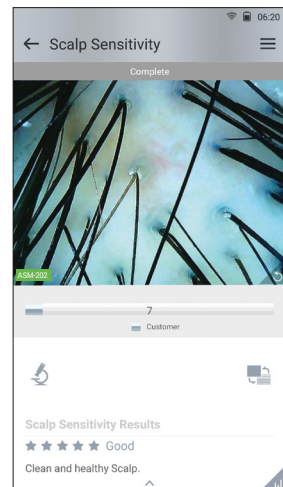


6 Check the result. To move on next page, slide the screen from right to left.

## Taking off the lens ASM-203

- 1 Press switching button located below the lens for switching the lens.

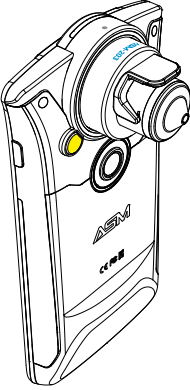


- 2 Turn the lens about 45 degrees counter-clockwise while pressing the switching button to take off the lens.

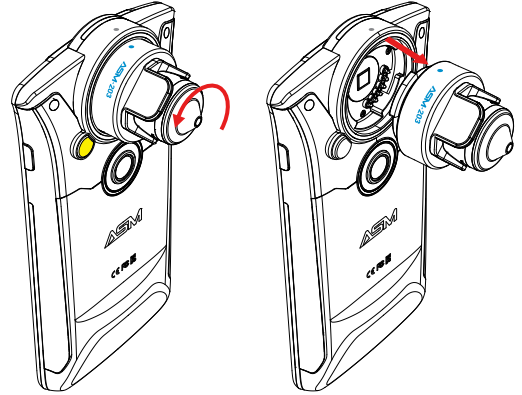


## Inserting lens


- 1 Until the correct lens is inserted, lens indicating label will blink.

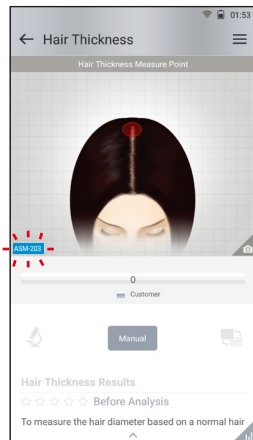


- 2 To insert the lens, match the red dot on lens and gray dot on device, turn it clockwise until clicks.

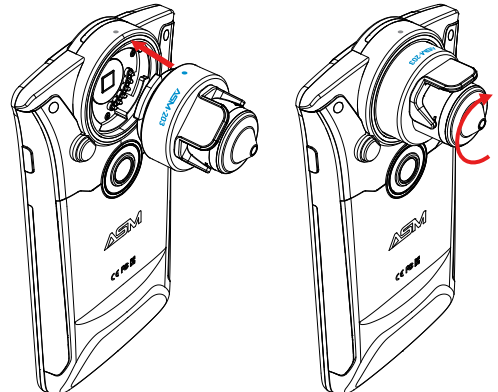

# Hair Thickness ASM-203

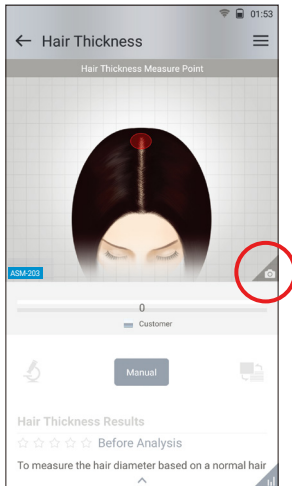


① Check capturing part and press  button to get into capture mode.

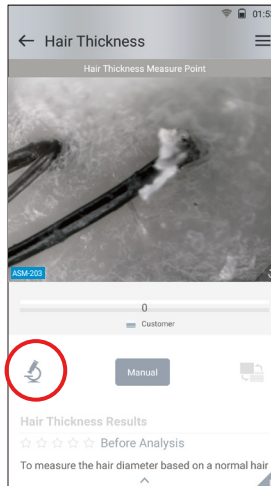


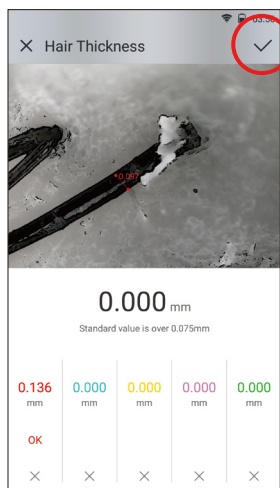
② Focus on image which you wish to capture.





③ Draw a line from one end to the other end on hair from enlarged captured image.

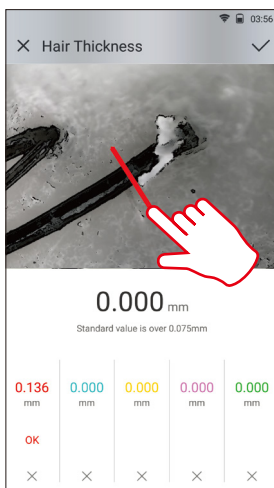

④ If select  from ②, average value will be calculated from total of 5 trials.

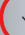


⑤ Click  to finish analysis and save.

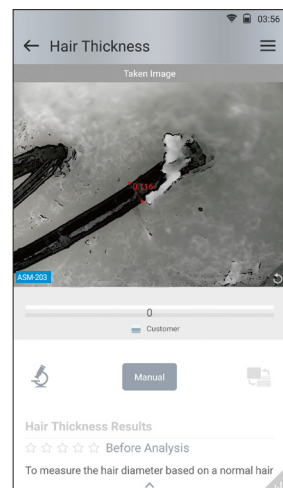


⑥ Check the result. To move on next page, slide the screen from right to left.


# Hair Pore Status ASM-203

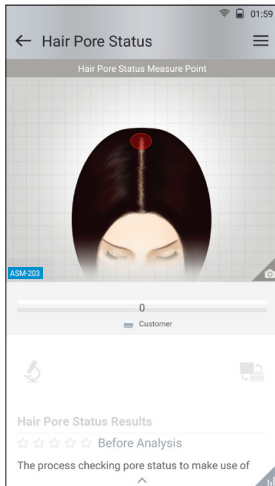


① Check capturing part and press  button to get into capture mode.

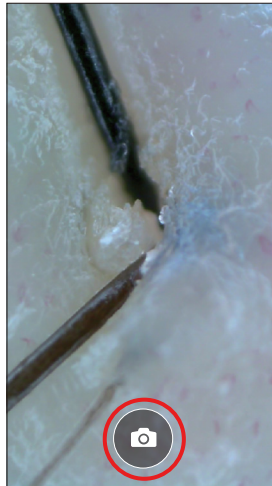


② Focus on image which you wish to capture.

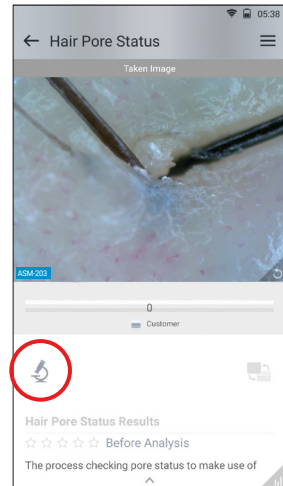


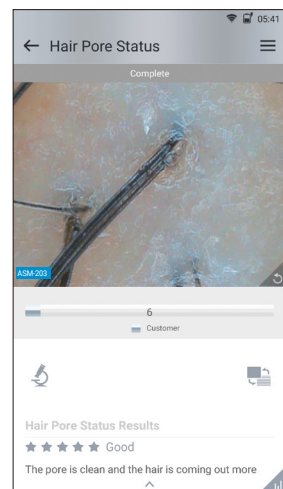
③ Press  button to analyze.



④ Select type and sample image similar to captured image.

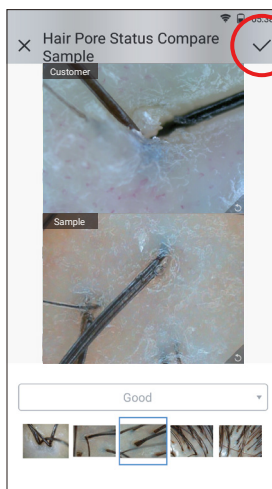


⑤ Click  to finish analysis and save.




⑥ Check the result. To move on next page, slide the screen from right to left.

## Taking off the lens ASM-204

- 1 Press switching button located below the lens for switching the lens.

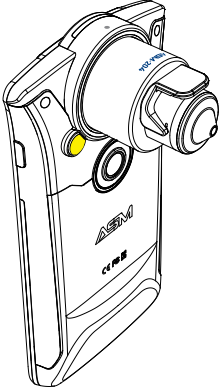


- 2 Turn the lens about 45 degrees counter-clockwise while pressing the switching button to take off the lens.

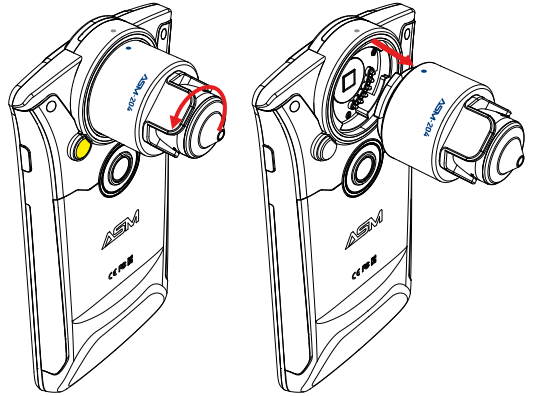


## Inserting lens


- 1 Until the correct lens is inserted, lens indicating label will blink.

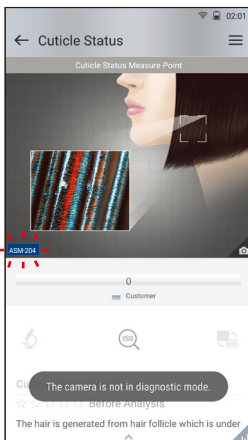


- 2 To insert the lens, match the red dot on lens and gray dot on device, turn it clockwise until clicks.

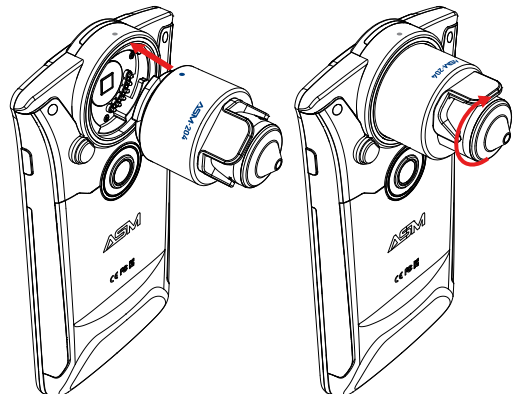


# Cuticle Status

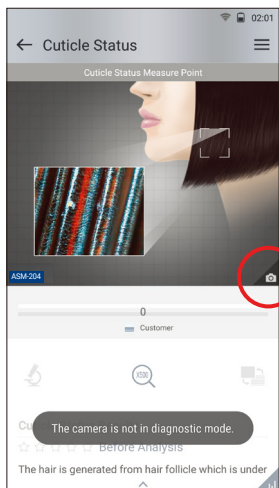


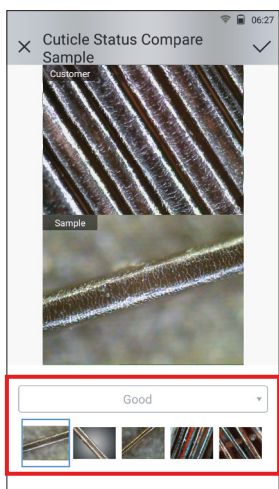
1 Check capturing part and press  button to get into capture mode.

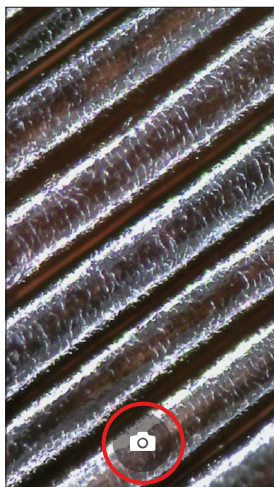


2 Focus on image which you wish to capture.

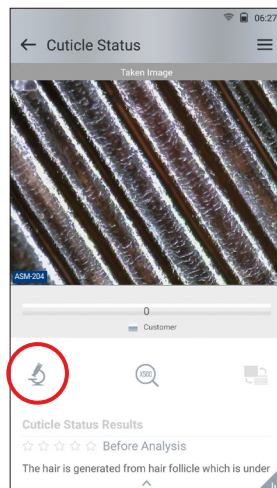



3 Press  button to analyze.



4 Select type and sample image similar to captured image.

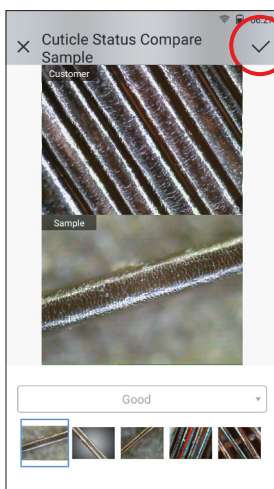


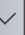5 Click  to finish analysis and save.

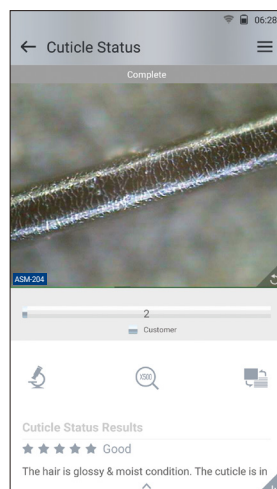


6 Check the result. To move on next page, slide the screen from right to left.


# Summary

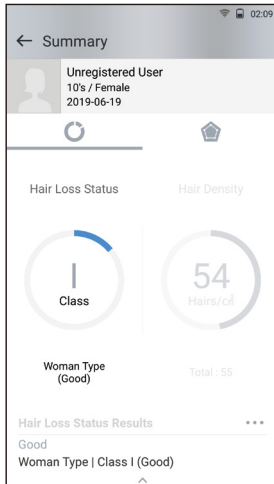

- ① The results will be shown in two pages.

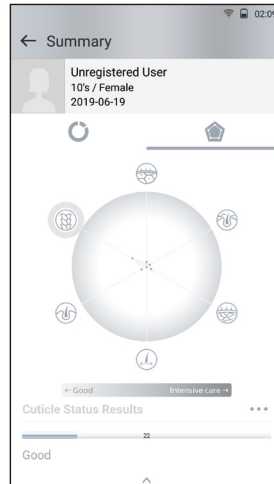

- ② On second page, the description of each parameter would be available by clicking ^ icon.

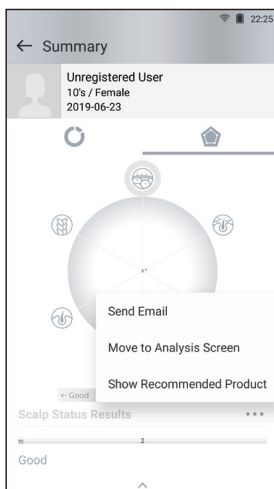


- ③ Once you select "Show Recommended Product", product image will pop up as ③.




- ④ Available to check product image.


# Send Result - Email

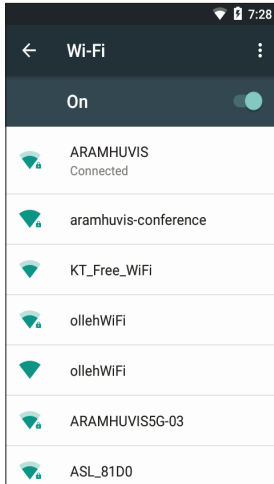


1 Before sending Email, the device shall be connected to Wi-Fi.

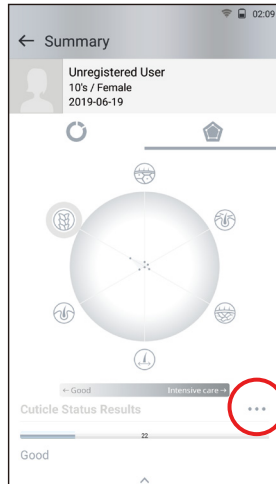


2 Touch ...

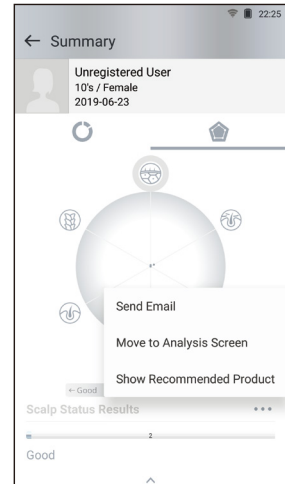


3 Go to "Send Email".

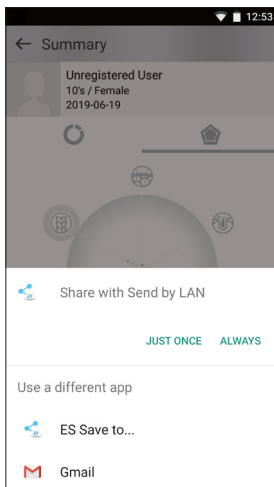

4 Select "Gmail"

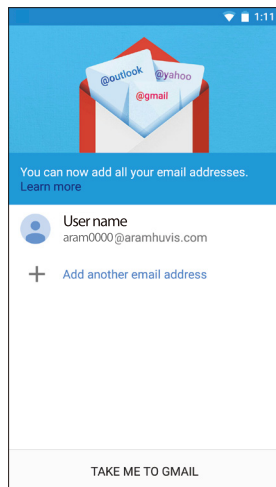


5 Select email account.

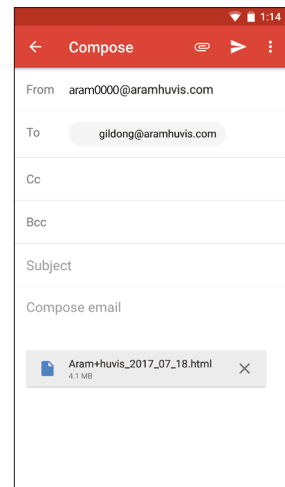


6 Share the result by sending email.




# Mirroring

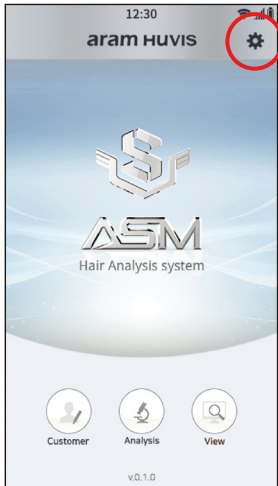

① Touch  button.

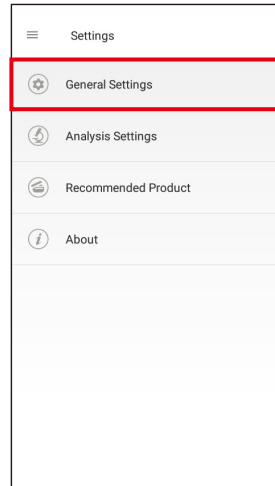

② Go to "General Settings".

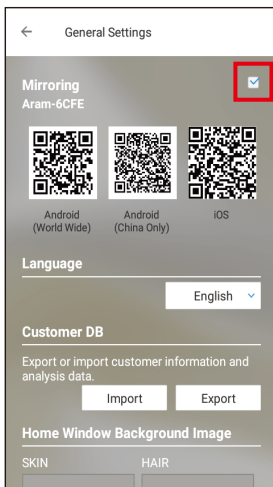


③ Turn on mirroring option.



④ Transfer to app download page using QR code.


# Mirroring

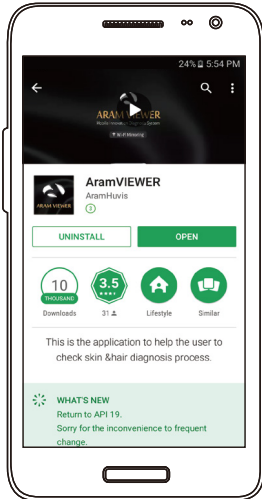

- 5 Install "Aram Viewer" on device which you will be mirroring.



- 6 Move to "Wi-Fi" setting.

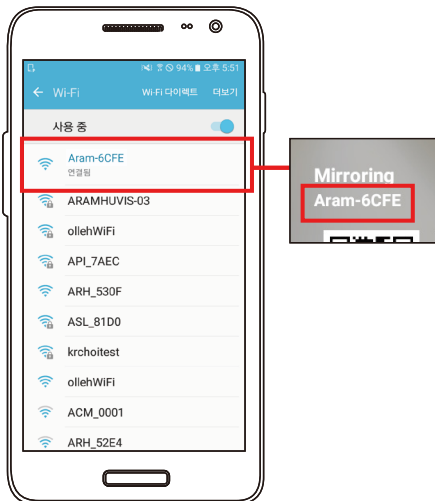

- 7 Select device's SSID.




- 8 Mirroring will be started.

# View Mode




① Touch "View".

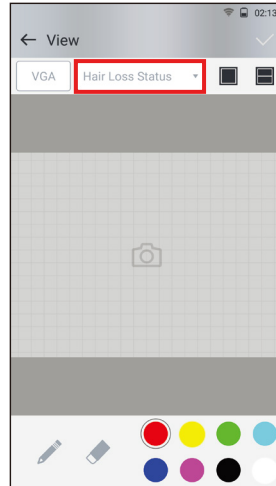

② You can analyze with multiple modes.



③ You can compare 2 pictures.

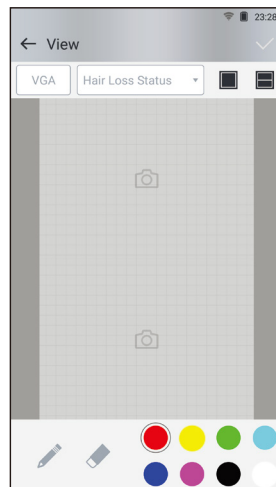

④ You can mark on screen using pen.

# Warranty

- ① The product undergoes strict quality management and inspection process before production and introduction to market.
- ② In case of functional or operational defect occurs under normal condition of use within 10 days after purchase, we will repair it for free or replace in new product.
- ③ When this product goes out under the normal use, we will repair it free of charge under warranty.
- ④ Please keep the warranty card when you ask for repair service.
- ⑤ Customer needs to pay repair charge after warranty runs out.
- ⑥ For this product, warranty period is 14 months from the Invoice date.
- ⑦ Cost
  - Within warranty : Sending delivery(Buyer), repair cost and returning delivery(Aram)
    - If the product is broken by user's fault, repair cost could be charged.
    - After warranty : Sending delivery and repair cost(Buyer), returning delivery(Aram)
- ⑧ How to send :
  - As sending a present from person to person by EMS or FEDEX.  
(Do not write the company name)
  - Total value should be under US \$ 99.(Write non-commercial value)

# Warranty

Service request form			
Date of purchase	/	/	Company
Contact person			
Address :			
Tel :			
Product name, serial No :			
Problems :			

SHIPPING ADDRESS :

#801 Byucksan Technopia, 560, Dunchon-daero, Jungwon-gu, Seongnam-si,  
Gyeonggi-do, 13230 Republic of Korea

Tel: 070-4687-1712 / Fax: 031-732-6330 / Email: support@aramhuvis.com  
<http://www.aramhuvis.com>

# Memo

---

# Memo

---



**HEAD OFFICE**

#401~402 Bundang Seoul National University Hospital Health Care Innovation Park, 172, Dolma-ro, Bundang-gu, Seongnam-si, Gyeonggi-do, 13605 Republic of Korea

**PRODUCTION DIVISION & FACTORY**

#909 Byucksan Technopia, 560, Dunchon-daero, Jungwon-gu, Seongnam-si, Gyeonggi-do, 13230 Republic of Korea

Tel: 070-4687-1712 / Fax: 031-732-6330 / Email: [info@aramhuvis.com](mailto:info@aramhuvis.com)  
<http://www.aramhuvis.com>