


ASW-100

User Manual

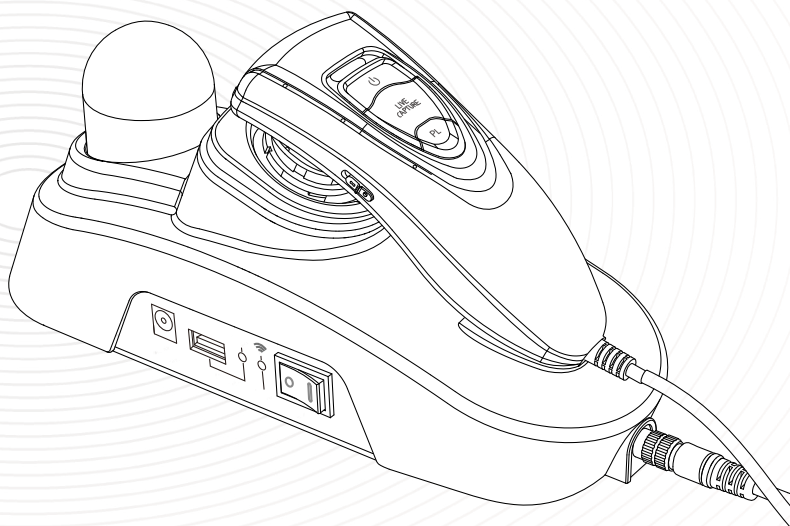


Contents

A. Components	2
<hr/>	
B. H/W Parts	
i. Both Sides	3
ii. Settings	4
<hr/>	
C. System Settings	
i. Download&Install	5
ii. How to connect with the program	6
iii.USB Connection / Automatic focusing lens set	9
<hr/>	
D. Program Settings	10
<hr/>	
E. Application	
i. Start Analysis	11
ii. Full Analysis	
CRM	12
Moisture	13
Sebum	14
Pore	15
Melanin	16
Acne	17
Wrinkle	18
Sensitivity	19
Summary	20
iii. View Mode	21
<hr/>	
F. Warranty	22


※ Notes on use

- The images of this manual can be different with the actual things.
- The information contained in this manual and future improvements are subject to change without notice.
- The latest version of the manual will be updated at the below link.
<http://www.aramhuvis.com/download>

Components

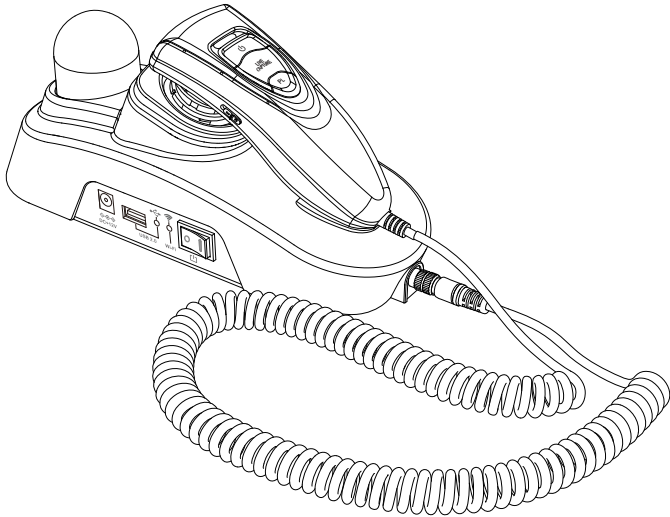


Main body with camera

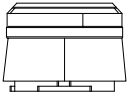

Lens x1



Lens x30

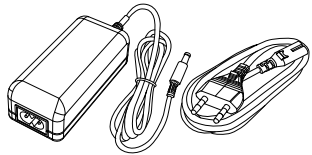

Charger set



Air blower



Sebum paper 50ea




Cleaning cloth



Manual

H/W Parts

Both Sides

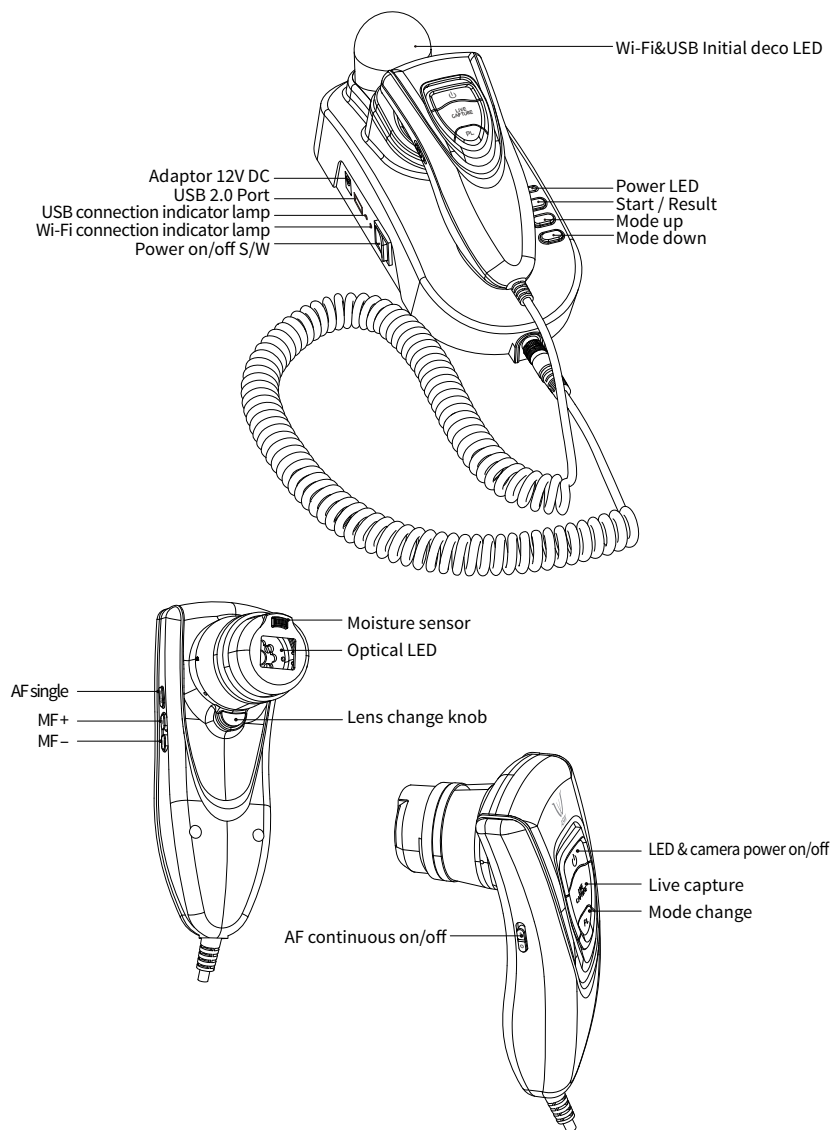


H/W Parts

Settings


To turn on & off the power

- 1 Switch on the power.
- 2 LED will be turned in blue light. While the wi-fi is turning on, the light twinkles and once it stops, the device is ready to use.

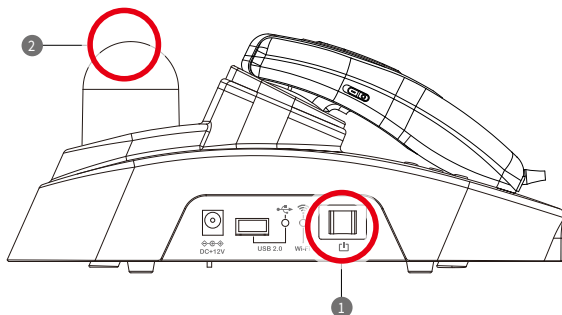

To replace lens

- 1 Press the button.
- 2 Then insert the lens into the hole and turn to right.

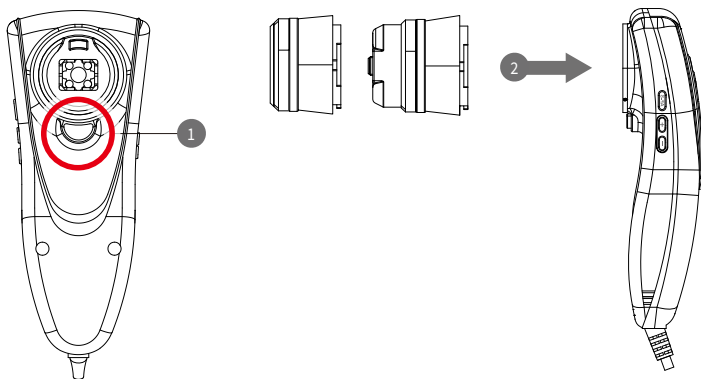

System Settings

Download & Install

PC User


- 1 Go to link on the browser as below.
<http://www.aramhuvis.com/download>
- 2 Install "Wizard" program.

Tablet device or Smartphone user



- 1 iOS : Go to "App Store" and search "aramhuvis" then download "Wizard-AramHuvis" application.
- 2 Android : Go to "Google Play Store" and search "aramhuvis" then download "Wizard-AramHuvis" application

System Settings

How to connect with the program

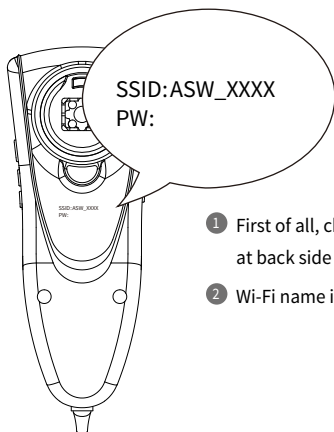


- 1 First of all, check the Wi-Fi name and password which is described at back side of camera.
- 2 Wi-Fi name is "ASW_XXXX".

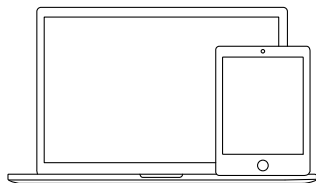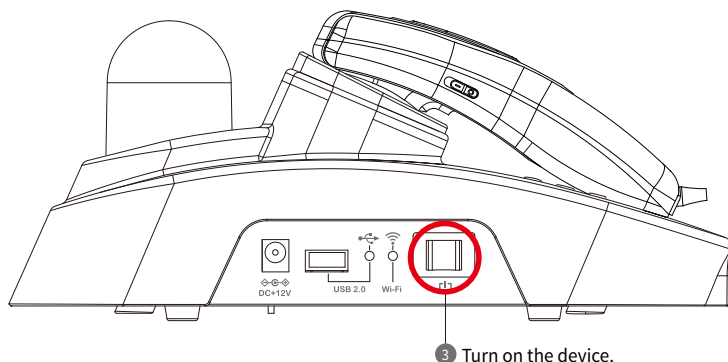


- 4 While turning on the device, prepare PC, Tablet to run the S/W.

System Settings

How to connect with the program

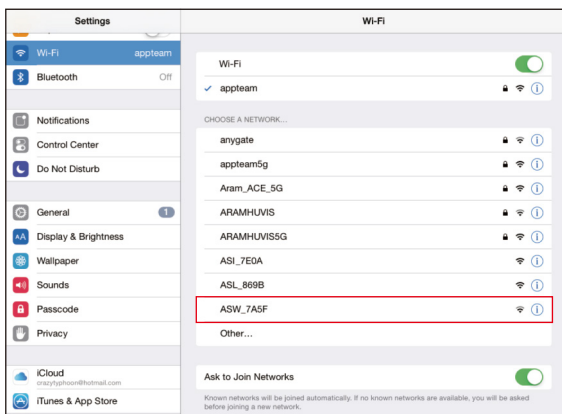

- 5 Once “ASW” device is turned on, please go to network setting to set up the Wi-Fi connection.

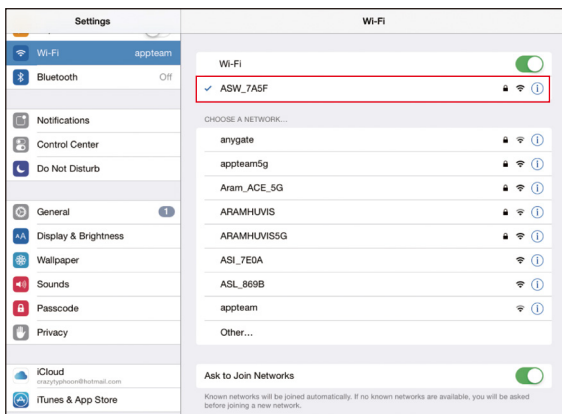

- 6 Among Wi-Fi lists, find “ASW-XXXX” and select it.

System Settings

How to connect with the program



7 When above code is connected, run the program "Wizard".



8 To check working properly, please check the program and see the left below side.


9 If it is correctly set up, you can see the beige light of Wi-Fi signal.

System Settings

USB Connection

USB

- PC: Wi-Fi & USB simultaneous support
- For PC, you can use the program by connecting the cable to wifi or usb port.

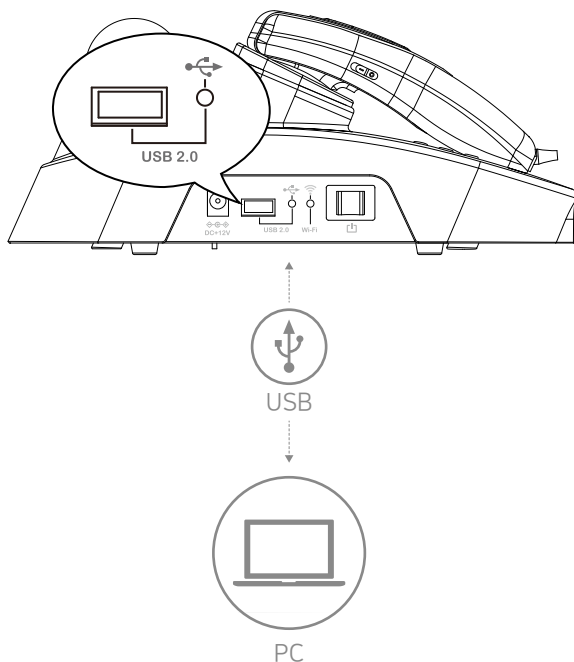

• Automatic Focusing Lens Set •




X 1 Lens :
Customer face




X 30 Lens :
Moisture / Sebum / Pore / Melanin
Acne / Wrinkle / Sensitivity

Program Settings

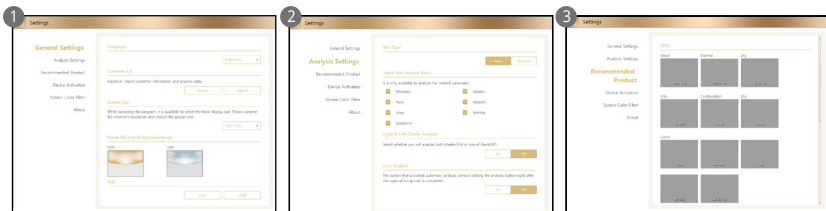

- 1 General settings**
Change the language, analysis mode and etc.
- 2 Analysis settings**
Set up the skin type and analysis parameter.
- 3 Recommended products**
Available for each analysis result.

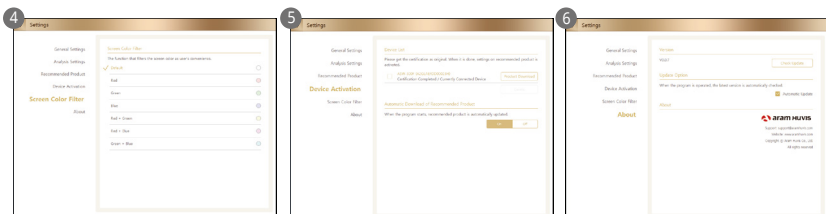


- 4 Color filter**
It is a function that calibrates the screen color according to the user's convenience.
- 5 Device Activation**
With the model name and serial code, you can activate your device and download recommended products.
- 6 About**
Check the current version and update is available in Wi-Fi connection.

Application


Start Analysis

- 1 Select "Analysis" to start.

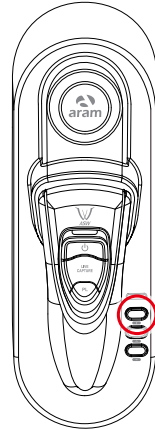


Remote Control

- 1 During analysis, you can control the device in "Wizard" program.
- 2 You can make use of this function in all analysis pages.

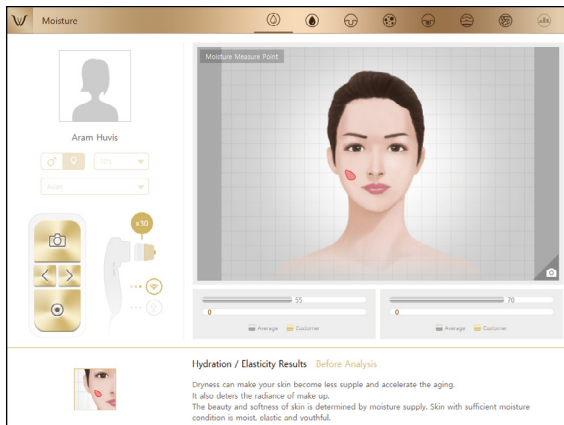


Application


CRM

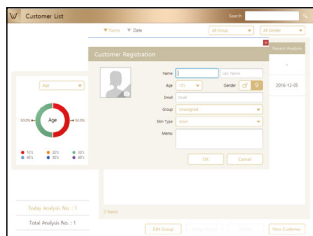


- 1 Input the information.
- full name, email address, gender and age group

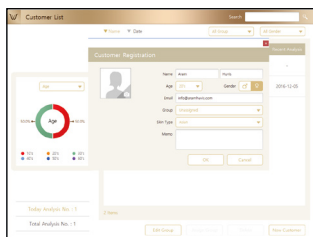

- 2 Take picture of customer's face with built-in camera or use the picture you already have in device.

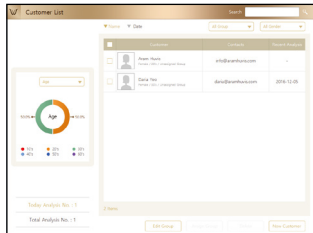


- 3 Among the customer lists, select the client to check or start analysis.

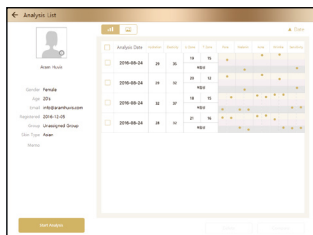

- 4 Check the customer information and analysis result list.

Application


Moisture

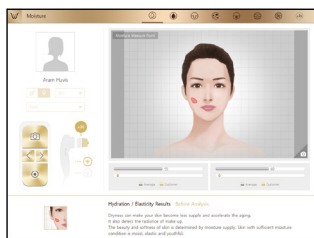


- 1 Press "live capture" button and when you hear "beep" sound, please put the sensor onto your skin and wait until you hear "beep" sound again.




Moisture sensor

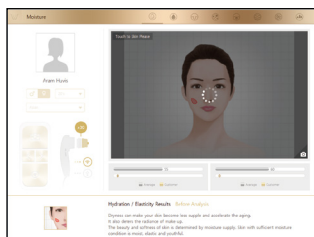

- 2 Analyzing hydration...

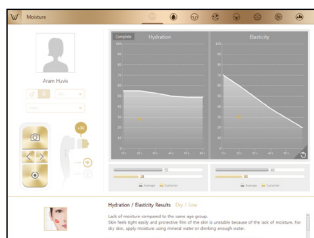


- 3 Check the result.
To analyze next step, select > button or press "PL" or <- button.

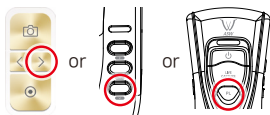

Application


Sebum

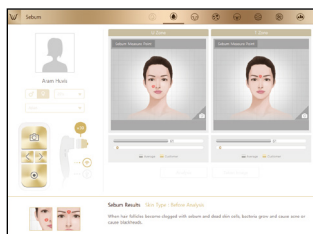

- 1 Put the sebum paper onto the U-zone on the cheek, then press it for 2~3 seconds.




U-zone

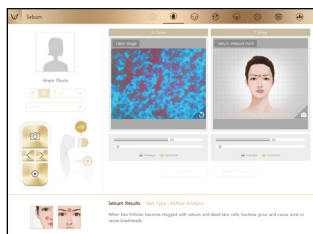

- 2 Place camera on the pink area and take picture. (U-zone)



Sebum paper

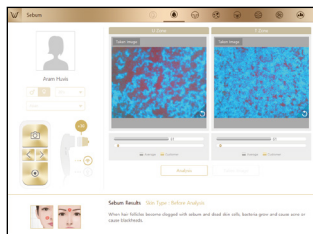


- 3 Put the sebum paper onto the T-zone on face, then press it for 2~3 seconds. Place camera on the pink area and take picture.(T-zone)






T-zone

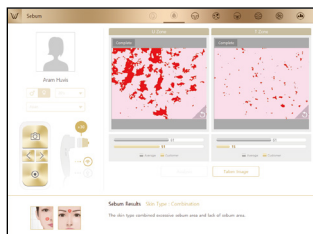

Sebum paper





- 4 Check the result. To analyze next step, select  button or press "PL" or  button.

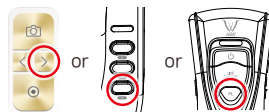


Application



Pore

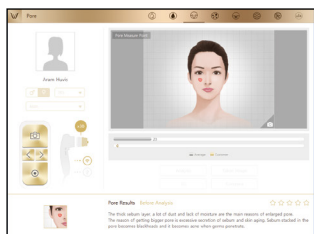


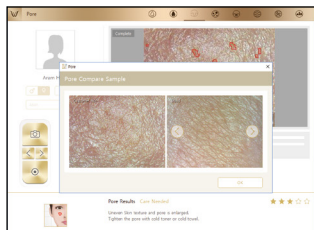
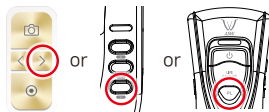
- 1 Select  button to activate camera or "live capture" button to activate camera mode.

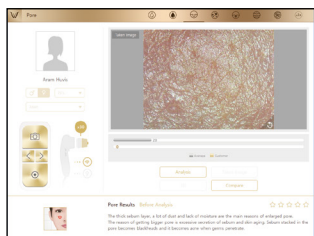

- 2 Press "live capture" button to take picture and press it again to analyze.

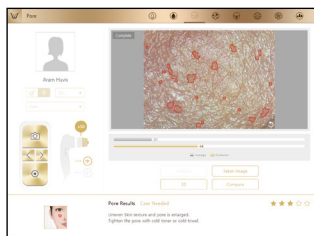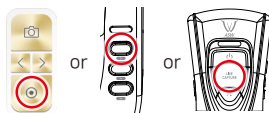

- 3 The result will appear.
Select "Compare" button to check the sample images as .
To analyze the next step, select the below button.





- 4 Compare your image with 3 sample images.


- Good
- Normal
- Bad

Application



Melanin

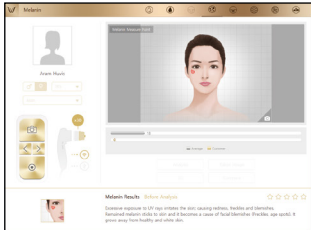


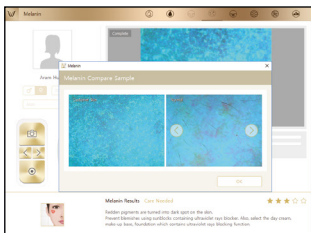
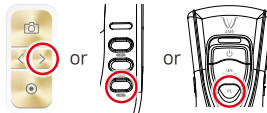
- 1 Select  button to activate camera or "live capture" button to activate camera mode.

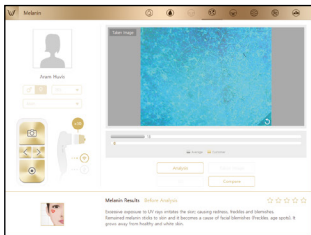

- 2 Press "live capture" button to take picture and press it again to analyze.

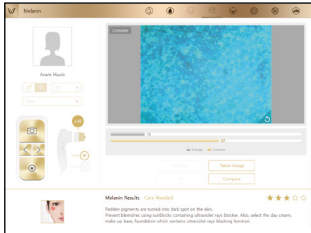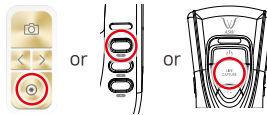

- 3 The result will appear. Select "Compare" button to check the sample images as . To analyze the next step, select the below button.





- 4 Compare your image with 3 sample images.


- Good
- Normal
- Bad

Application



Acne

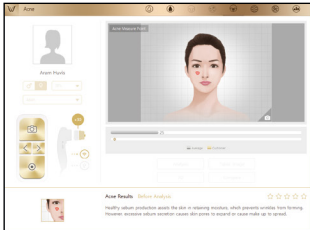



- 1 Select  button to activate camera or "live capture" button to activate camera mode.

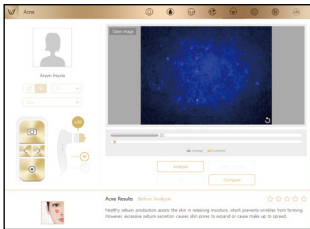

- 2 Press "live capture" button to take picture and press it again to analyze.

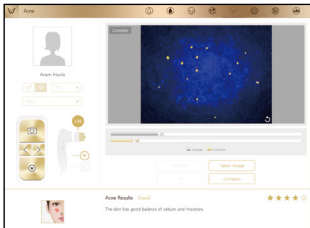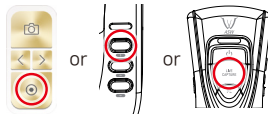

- 3 The result will appear. Select "Compare" button to check the sample images as 4. To analyze the next step, select the below button.

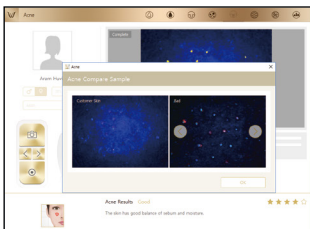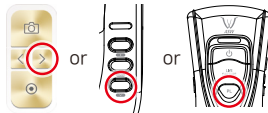



- 4 Compare your image with 3 sample images.


- Good
- Normal
- Bad

Application



Wrinkle

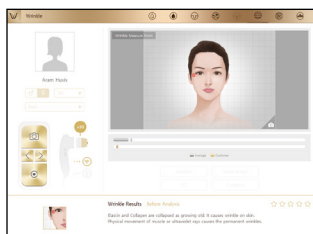


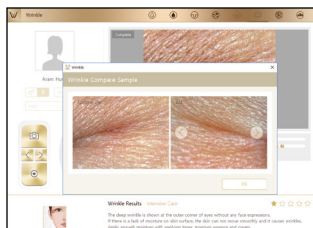
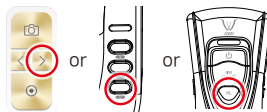
- 1 Select  button to activate camera or "live capture" button to activate camera mode.

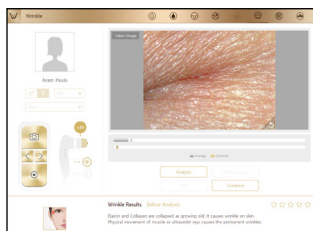

- 2 Press "live capture" button to take picture and press it again to analyze.

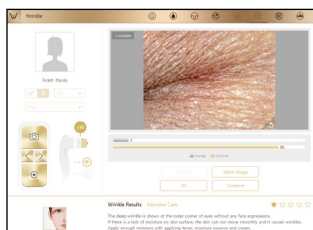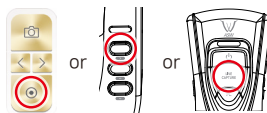


- 3 The result will appear. Select "Compare" button to check the sample images as . To analyze the next step, select the below button.




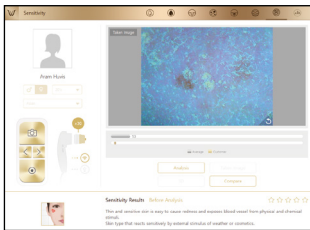
- 4 Compare your image with 3 sample images.
 - Good
 - Normal
 - Bad

Application



Sensitivity

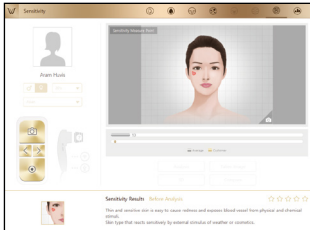



- 1 Select  button to activate camera or "live capture" button to activate camera mode.



- 2 Press "live capture" button to take picture and press it again to analyze.

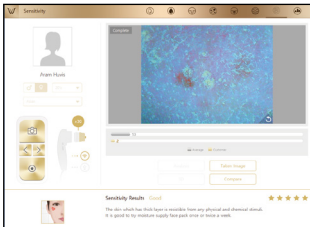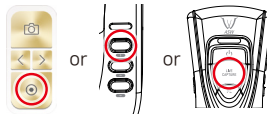

- 3 The result will appear. Select "Compare" button to check the sample images as **4**. To analyze the next step, select the below button.

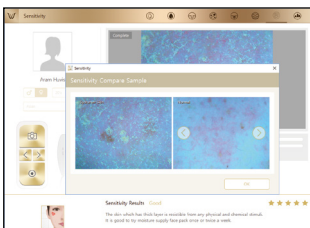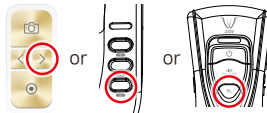


- 4 Compare your image with 3 sample images.


- Good
- Normal
- Bad

Application

Summary

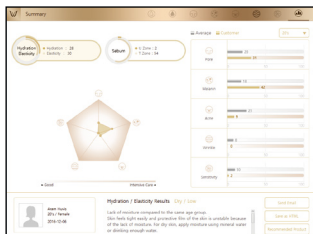


- 1 The diagnostic icon in the pentagon or the name of each diagnostic select to see the results below. If you select a recommended product, you can see the product.

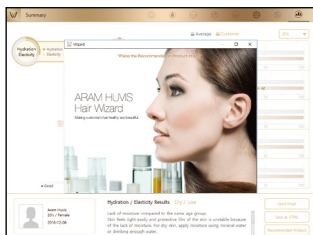


- 2 Available to check product image.

Application


View Mode

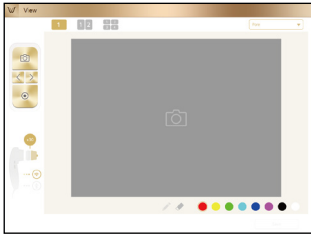


- 1 In view mode, it is able to take pictures with capture button in many times.

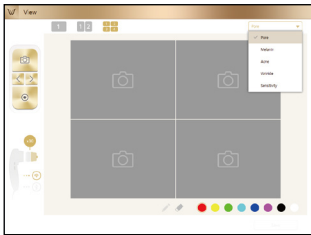

- 2 The user can select analysis parameter to take picture.

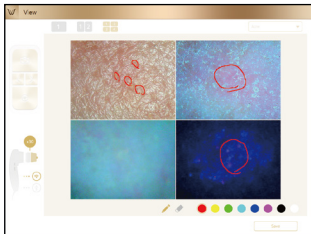

- 3 In one screen, you can see 4 different images maximum.

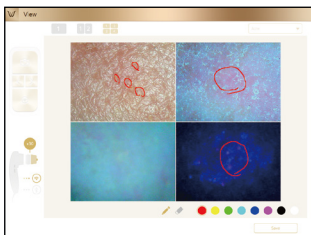

- 4 The user can mark the defect or concern according to the image with note function.

Warranty

- 1 The product undergoes strict quality management and inspection process before production and introduction to market.
- 2 In case of functional or operational defect occurring under normal condition of use within 10 days after purchase, we will repair it for free or replace in new product.
- 3 When this product goes out under the normal use, we will repair it free of charge under warranty.
- 4 Please keep the warranty card when you ask for repair service.
- 5 Customer needs to pay repair charge after warranty runs out.
- 6 For this product, warranty period is 14 months from the Invoice date.
- 7 Cost
 - Within warranty : Sending delivery(Buyer), repair cost and returning delivery(Aram)
 - If the product is broken by user's fault, repair cost could be charged.
 - After warranty : Sending delivery and repair cost(Buyer), returning delivery(Aram)
- 8 How to send :
 - As sending a present from person to person by EMS or FEDEX.
 - (Do not write the company name)
 - Total value should be under US \$ 99.(Write non-commercial value)

Warranty

Service request form			
Date of purchase	/ /	Company	
Contact person			
Address :			
Tel :			
Product name, serial no :			
Problems :			

Shipping address:

801-ho , 560 Dunchon-daero, Jungwon-gu Seongnam-si Gyeonggi-do 13230 KOREA.

E-mail : support@aramhuvis.com

Tel : +82 70 4687 1712 / Fax : +82 31 732 6330

MEMO

MEMO

MEMO



HEAD OFFICE

#401~402 Bundang Seoul National University Hospital Health Care Innovation Park, 172, Dolma-ro, Bundang-gu, Seongnam-si, Gyeonggi-do, 13605 Rep. Korea

PRODUCTION DIVISION & FACTORY

#909 Byucksan Technopia, 560, Dunchon-daero, Jungwon-gu, Seongnam-si, Gyeonggi-do, 13230 Rep. Korea

Tel: 070-4687-1712 / Fax: 031-732-6330 / Email: info@aramhuvis.com
<http://www.aramhuvis.com>